

Step 4: Download DICOMs or NIFTI

a) Open MR session:

PROJECT_01303 > 20201203-35213

Subject Details: 20201203-35213

Details	Projects
Accession # XNAT_ZIH2_S00583 Date Added 2020-12-04 11:27:56.569 (mame720c) Birth year -- Gender Handedness	

Actions

- Edit
- View XML
- Add Experiment
- Download XML
- Email
- Manage Files
- Delete

Experiments

Date	Experiment	Project	Label
2020-12-03	MR Session	01303 NICQA d32 FBIRN	16363_1

b) Click on 'Download Images'

PROJECT_01305 > SUBJECT_20190720 > 13914_1

MR Session: 13914_1

Details	Projects
Accession #: XNAT_ZIH02_E01354 Date Added: 12/09/2019 11:44:32 (rbaensch) Date: 07/20/2019 Time: 09:04:38 Scanner Name: MRC35213 Scanner Type: SIEMENS TrioTim Acquisition Site: NIC	Subject: 20190720 Gender: Handedness: Age: --

Actions

- Edit
- Build
- View
- Download
- Email
- Manage Files
- View Images
- Delete

Scans

Scan	Type	Series Desc	Usability	Files	Note
<input type="checkbox"/> 1	ep2d_bold_rsMRT_Phantom	ep2d_bold_rsMRT_Phantom	usable	90.3 MB in 250 files	
<input type="checkbox"/> 2	rf_noise	rf_noise	usable	10.0 MB in 50 files	
<input type="checkbox"/> 3	spike	spike	usable	754.1 KB in 2 files	

Total: 101.1 MB in 302 files

History

c) Click on Download

